# Installation Manual

## RadiForce® MX242W

Color LCD Monitor

### Important

Please read this Installation Manual, and “Instructions for Use” (separate volume) carefully to familiarize yourself with safe and effective usage.

- 
- For monitor adjustment and settings, refer to the Installation Manual.
  - The latest “Instructions for Use” is available for download from our web site:

<http://www.eizo.com>

---

This product has been adjusted specifically for use in the region to which it was originally shipped. If operated outside this region, the product may not perform as stated in the specifications.

No part of this manual may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, or otherwise, without the prior written permission of EIZO Corporation. EIZO Corporation is under no obligation to hold any submitted material or information confidential unless prior arrangements are made pursuant to EIZO Corporation's receipt of said information. Although every effort has been made to ensure that this manual provides up-to-date information, please note that EIZO monitor specifications are subject to change without notice.

# CONTENTS


<b>CONTENTS</b> .....	<b>3</b>	<b>5-3. Imaging Problems (for analog only)</b> .....	<b>22</b>
<b>Chapter 1 Basic Operations and Functions</b> ....	<b>4</b>	<b>5-4. Other Problems</b> .....	<b>23</b>
<b>1-1. Basic Adjustment menu operations</b> .....	<b>4</b>	<b>Chapter 6 Reference</b> .....	<b>24</b>
<b>1-2. Showing Button Guide</b> .....	<b>5</b>	<b>6-1. Attaching Arm</b> .....	<b>24</b>
<b>1-3. Functions</b> .....	<b>6</b>	<b>6-2. Detaching/Attaching Stand Base</b> .....	<b>25</b>
<b>Chapter 2 Adjusting Screens</b> .....	<b>7</b>	<b>6-3. Attaching/Detaching Cable Holder</b> .....	<b>28</b>
<b>2-1. Compatible Resolutions</b> .....	<b>7</b>	<b>6-4. Connecting Multiple PCs</b> .....	<b>30</b>
<b>2-2. Setting the Resolution</b> .....	<b>7</b>	● Switching among input signals.....	<b>30</b>
● Windows 8.1 / Windows 8 / Windows 7.....	<b>7</b>	● Setting input signal switching procedure .....	<b>31</b>
● Windows Vista.....	<b>7</b>	<b>6-5. Making Use of USB</b>	
● Windows XP .....	<b>7</b>	<b>(Universal Serial Bus)</b> .....	<b>32</b>
● Mac OS X .....	<b>7</b>	● Required system environment .....	<b>32</b>
<b>2-3. Adjusting Color</b> .....	<b>8</b>	● Connection procedure	
● Selecting the display mode		(Setup of USB function).....	<b>32</b>
(CAL Switch mode) .....	<b>8</b>	<b>6-6. Showing Monitor Information</b> .....	<b>33</b>
● Performing advanced adjustments .....	<b>8</b>	● Displaying signal information .....	<b>33</b>
● Adjustable settings in each mode .....	<b>9</b>	● Showing monitor information .....	<b>33</b>
● Adjusting Brightness .....	<b>9</b>	<b>6-7. Main Default Settings</b> .....	<b>34</b>
● Adjusting Temperature .....	<b>10</b>	<b>6-8. Preset Timing</b> .....	<b>35</b>
● Adjusting Gamma .....	<b>10</b>	<b>LIMITED WARRANTY</b> .....	<b>36</b>
● Adjusting Hue .....	<b>10</b>		
● Adjusting Saturation .....	<b>11</b>		
● Adjusting Gain .....	<b>11</b>		
<b>2-4. Adjusting Screens</b> .....	<b>12</b>		
● Digital Signal Input .....	<b>12</b>		
● Analog Signal Input .....	<b>12</b>		
<b>2-5. Selecting Screen Size</b> .....	<b>15</b>		
<b>Chapter 3 Setting Monitor</b> .....	<b>16</b>		
<b>3-1. Skipping Unused Display Modes</b> .....	<b>16</b>		
<b>3-2. Setting Language</b> .....	<b>16</b>		
<b>3-3. Setting Adjustment Menu Display</b> .....	<b>16</b>		
<b>3-4. Showing and hiding the EIZO logo</b> .....	<b>17</b>		
<b>3-5. Locking Operation Buttons</b> .....	<b>17</b>		
<b>3-6. Enabling/Disabling DDC Communication</b> .....	<b>17</b>		
<b>3-7. Resetting to Default</b> .....	<b>18</b>		
● Resetting the color adjustments .....	<b>18</b>		
● Resetting all adjustments to the default			
settings .....	<b>18</b>		
<b>Chapter 4 Power Saving Functions</b> .....	<b>19</b>		
<b>4-1. Setting Power Save</b> .....	<b>19</b>		
<b>4-2. Adjusting the Brightness of Indicator</b> .....	<b>19</b>		
<b>4-3. Setting DisplayPort Power Save</b> .....	<b>20</b>		
<b>Chapter 5 Troubleshooting</b> .....	<b>21</b>		
<b>5-1. No Picture</b> .....	<b>21</b>		
<b>5-2. Imaging Problems</b>			
<b>(for both digital and analog)</b> .....	<b>22</b>		

# Chapter 1 Basic Operations and Functions

## 1-1. Basic Adjustment menu operations


### 1. Displaying Adjustment menu

1. Press ENTER. The Adjustment menu appears.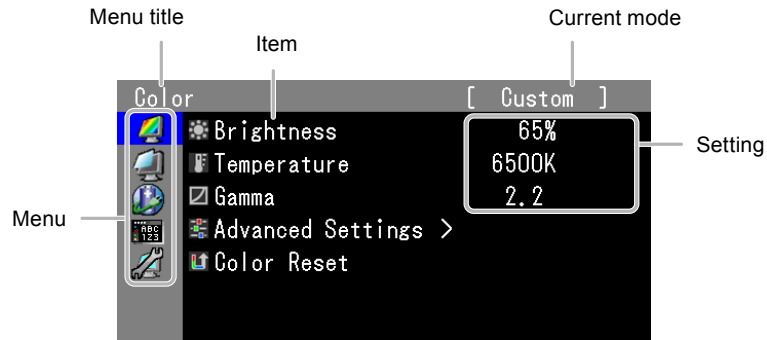


### 2. Adjusting/Setting


1. Select a menu to adjust/set with ▲ or ▼, and then press ENTER.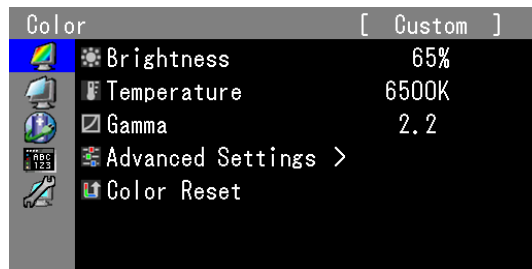

2. Select an item to adjust/set with ▲ or ▼, and then press ENTER.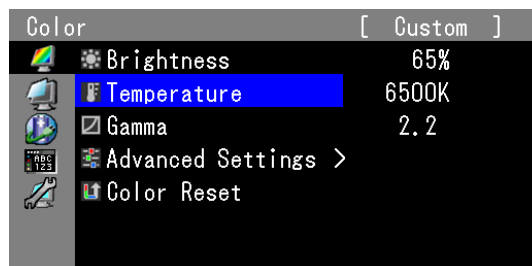

3. Adjust/set the selected item with ▲ or ▼, and then press ENTER.

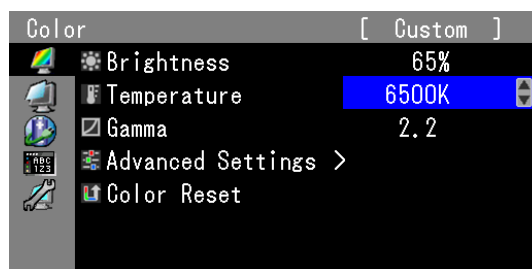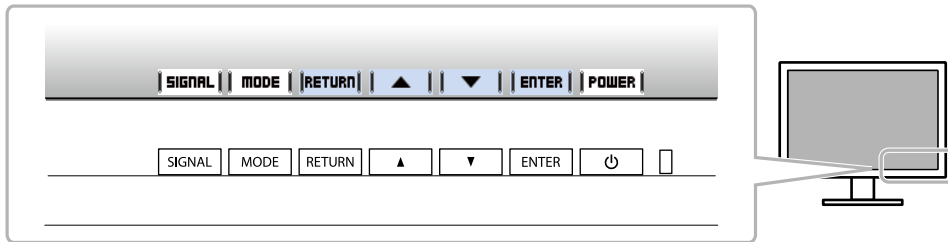
### 3. Exiting the software

1. Press RETURN a few times. The Adjustment menu finishes.

## 1-2. Showing Button Guide

---

When you press the front buttons (except the  button), the button guide appears next to the button.

---

### Note

- The button guide will continue to appear while the Adjustment menu or Mode menu is showing.
  - The button guide is displayed differently depending on the selected menu or status.
-

# 1-3. Functions

The following table shows the Adjustment menu and setting items of each menu.

Main Menu	Item	Reference	
Color <sup>*1</sup> 	Brightness	"2-3. Adjusting Color" (page 8)	
	Temperature		
Gamma			
Advanced Settings	Hue		
Saturation			
	Gain		
	Color Reset	"3-7. Resetting to Default" (page 18)	
Screen 	Screen Size	"2-5. Selecting Screen Size" (page 15)	
	Analog Adjustment <sup>*2</sup>	Auto Adjustment	"2-4. Adjusting Screens" (page 12)
		Range Adjustment	
		Clock	
		Phase	
		Hor. Position	
		Ver. Position	
Power Manager 	Power Save	"4-1. Setting Power Save" (page 19)	
	Indicator	"4-2. Adjusting the Brightness of Indicator" (page 19)	
Menu Settings 	Language	"3-2. Setting Language" (page 16)	
	Menu Position	"3-3. Setting Adjustment Menu Display" (page 16)	
Tools 	Input Selection	"Switching among input signals" (page 30)	
	Mode Skip	"3-1. Skipping Unused Display Modes" (page 16)	
	Signal Info	"6-6. Showing Monitor Information" (page 33)	
	Monitor Info		
	All Reset	"3-7. Resetting to Default" (page 18)	
Optional Settings	Logo	"3-4. Showing and hiding the EIZO logo" (page 17)	
	Key Lock	"3-5. Locking Operation Buttons" (page 17)	
	DDC	"3-6. Enabling/Disabling DDC Communication" (page 17)	
	DP Power Save	"4-3. Setting DisplayPort Power Save" (page 20)	

\*1 The adjustment/settings available on the "Color" menu depend on the selected mode (see "Adjustable settings in each mode" (page 9)).

\*2 These functions can be adjusted with analog input.

# Chapter 2 Adjusting Screens

## 2-1. Compatible Resolutions

For details on compatible resolutions, refer to “Compatible Resolutions” in the “Instructions for Use”.

## 2-2. Setting the Resolution

### ● Windows 8.1 / Windows 8 / Windows 7

1. For Windows 8.1 / Windows 8, click the “Desktop” tile on the Start Screen to display the desktop.
2. Right-click the mouse anywhere on the desktop except for icons.
3. From the displayed menu, click “Screen resolution”.
4. On the “Screen Resolution” dialog box, select the monitor.
5. Click “Resolution” to select the desired resolution.
6. Click the “OK” button.
7. When a confirmation dialog box is displayed, click “Keep changes”.

### ● Windows Vista

1. Right-click the mouse anywhere on the desktop except for icons.
2. From the displayed menu, click “Personalize”.
3. On the “Personalization” window, click “Display Settings”.
4. On the “Display Settings” dialog box, select the “Monitor” tab and select desired resolution in the “Resolution” field.
5. Click the “OK” button.
6. When a confirmation dialog box is displayed, click “Yes”.

### ● Windows XP

1. Right-click the mouse anywhere on the desktop except for icons.
2. From the displayed menu, click “Properties”.
3. When the “Display Properties” dialog box is displayed, click the “Settings” tab and select desired resolution for “Screen resolution” under “Display”.
4. Click the “OK” button to close the dialog box.

### ● Mac OS X

1. Select “System Preferences” from the Apple menu.
2. When the “System Preferences” dialog box is displayed, click “Displays” for “Hardware”.
3. On the displayed dialog box, select the “Display” tab and select desired resolution in the “Resolutions” field.
4. Your selection will be reflected immediately. When you are satisfied with the selected resolution, close the window.

## 2-3. Adjusting Color


### ● Selecting the display mode (CAL Switch mode)

A desired mode can be selected easily according to the monitor application.

#### Display Modes

√: Available, -: Not available

Mode	Purpose	Calibration
1-DICOM	Available to display in DICOM mode.	√
2-Custom	Available for the setting according to your preference.	√
3-CAL	Displays the screen adjusted by calibration software.	√
4-Text	Suitable for displaying texts for word processing or spreadsheets.	-

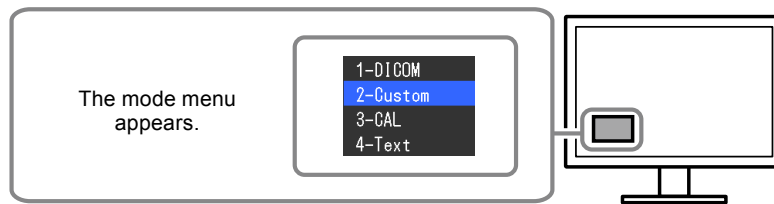#### Procedure

1. Press MODE.

The mode menu appears at the bottom left of the screen.

2. The mode among the list is highlighted in turn each time MODE is pressed.

You can switch the mode with ▲ or ▼ while the mode menu is displayed.

#### Note

- The Adjustment menu and the Mode menu cannot be displayed at the same time.

### ● Performing advanced adjustments

“Color” of the Adjustment menu allows you to configure and save a separate set of color settings for each mode.

#### Attention

- It takes about 30 minutes for the performance of electrical parts to stabilize. Please wait 30 minutes or more after the monitor power has been turned on or the monitor has recovered from the power saving mode, and then adjust the monitor.
- The same image may be seen in different colors on multiple monitors due to their monitor-specific characteristics. Make fine color adjustment visually when matching colors on multiple monitors.

#### Note

- The values shown in “%” or “K” are available only as reference.

## ● Adjustable settings in each mode

The adjustable settings depend on the mode. (Non-adjustable or non-settable function cannot be selected.)

√: Adjustable -: Not adjustable

Icon	Function	CAL Switch mode			
		1-DICOM	2-Custom <sup>*1</sup>	3-CAL	4-Text
	Brightness	√	√	√	√
	Temperature	-	√	-	√
	Gamma	-	√	-	√
	Hue	-	√	-	√
	Saturation	-	√	-	√
	Gain	-	√	-	-
	Color Reset	√	√	√	√

\*1 If you adjust the calibration in this mode using the calibration kit exclusive for to this monitor (see RadiCS LE /"Accessories" in the "instructions for Use"), only "Brightness" and "Color Reset" are adjustable.

## ● Adjusting Brightness

The screen brightness is adjusted by changing the brightness of the backlight (light source from the LCD back panel).

### **Setting Range**

0 to 100%

### **Procedure**

1. Select "Color" from the Adjustment menu, and press ENTER.
2. Select "Brightness" from "Color", and press ENTER.
3. Use ▲ or ▼ for adjustment.
4. Press ENTER to exit.

## ● Adjusting Temperature

The color temperature can be adjusted.

The color temperature is normally used to express the hue of “white” and/or “black” by a numerical value. The value is expressed in degrees “K” (Kelvin).

The screen becomes reddish at a low color temperature, and bluish at a high color temperature, like the flame temperature. The gain preset values are set for each color temperature setting value.

### **Setting Range**

Native, 6000K to 15000K (specified every 100K)

### **Procedure**

1. Select “Color” from the Adjustment menu, and press ENTER.
2. Select “Temperature” from “Color”, and press ENTER.
3. Use ▲ or ▼ for adjustment.
4. Press ENTER to exit.

---

#### **Note**

- “Gain” allows you to perform more advanced adjustment (see “Adjusting Gain” (page 11)).
  - If you set “Native”, the image is displayed in the preset color of the monitor (Gain: 100% for each RGB).
  - When gain is changed, the color temperature is changed to “User”.
- 

## ● Adjusting Gamma

Gamma can be adjusted. The brightness of the monitor varies depending on the input signal, however, the variation rate is not proportional to the input signal. To keep the balance between the input signal and the brightness of the monitor is called as “Gamma correction”.

### **Setting Range**

1.6 to 2.7

### **Procedure**

1. Select “Color” from the Adjustment menu, and press ENTER.
2. Select “Gamma” from “Color”, and press ENTER.
3. Use ▲ or ▼ for adjustment.
4. Press ENTER to exit.

## ● Adjusting Hue

The hue can be adjusted.

### **Setting Range**

-100 to 100

### **Procedure**

1. Select “Color” from the Adjustment menu, and press ENTER.
2. Select “Advanced Settings” from “Color”, and press ENTER.
3. Select “Hue”, and press ENTER.
4. Use ▲ or ▼ for adjustment.
5. Press ENTER to exit.

---

#### **Attention**

- After this adjustment, some color gradations may not be displayed.
-

## ● Adjusting Saturation

The color saturation can be adjusted.

### **Setting Range**

-100 to 100

### **Procedure**

1. Select “Color” from the Adjustment menu, and press ENTER.
2. Select “Advanced Settings” from “Color”, and press ENTER.
3. Select “Saturation”, and press ENTER.
4. Use ▲ or ▼ for adjustment.
5. Press ENTER to exit.

---

#### **Attention**

- After this adjustment, some color gradations may not be displayed.

---

#### **Note**

- The minimum value (-100) changes the screen to monochrome.
- 

## ● Adjusting Gain

The brightness of each color component red, green, and blue is called Gain. You can change the hue of “white” by adjusting gain.

### **Setting Range**

0 to 100%

### **Procedure**

1. Select “Color” from the Adjustment menu, and press ENTER.
2. Select “Advanced Settings” from “Color”, and press ENTER.
3. Select “Gain”, and press ENTER.
4. Select the color for adjustment from “Red”, “Green”, and “Blue”, and press ENTER.
5. Use ▲ or ▼ for adjustment.
6. Press ENTER to exit.

---

#### **Attention**

- After this adjustment, some color gradations may not be displayed.

---

#### **Note**

- The value of gain changes with that of color temperature.
  - When gain is changed, the color temperature is changed to “User”.
-

## 2-4. Adjusting Screens

---

### ● Digital Signal Input

When digital signals are input, images are displayed correctly based on the preset data of the monitor, but if you perform more advanced adjustment, see “2-3. Adjusting Color” (page 8) and subsequent pages.

### ● Analog Signal Input

---

**Attention**

- It takes about 30 minutes for the performance of electrical parts to stabilize. Please wait 30 minutes or more after the monitor power has been turned on or the monitor has recovered from the power saving mode, and then adjust the monitor.
  - The Self Adjust (automatic screen adjustment) function does not work for the images under the resolution of 800 × 600 (SVGA).
  - The self adjustment and auto adjustment functions work correctly when an image is fully displayed over the Windows or Macintosh display area. They do not work properly in the cases below:
    - when an image is displayed only on a part of the screen (command prompt window, for example)
    - when a black background (wallpaper, etc.) is in useAlso, these functions cannot work properly in some graphic boards.
- 

The monitor screen adjustment is used to suppress flickering of the screen or adjust screen position and screen.

---

**Note**

- The self adjustment function works when all of the following conditions are satisfied:
    - When a signal is input into the monitor for the first time or when a resolution or vertical/horizontal scan frequency not used before is
- 

If the screen is not displayed correctly even after performing the self adjustment operation, adjust the screen according to the procedures on the following pages to use the monitor comfortably.

### How to Set

#### 1. Perform the auto adjustment.

##### ● To adjust flickering, screen position, and screen size automatically

**Procedure**

1. Choose “Screen” from the Adjustment menu, and press ENTER.
2. Choose “Analog Adjustment” from “Screen”, and press ENTER.
3. Choose “Auto Adjustment” from “Analog Adjustment”, and press ENTER.

Flickering, screen position, and screen size are corrected by the auto-adjustment function.

If the screen is not displayed correctly even after performing Auto Adjustment, perform the adjustments according to the procedures on the following pages. If the screen is displayed correctly, go to “5. Adjusting color gradation.” (page 15).

#### 2. Prepare to use analog screen adjustment patterns.

Download the “Screen adjustment pattern files” from our site: <http://www.eizo.com>.

---

**Note**


- For details and instructions on opening the “Screen adjustment pattern files”, refer to the Readme.txt file.
-

### 3. Perform the auto adjustment again with the analog screen adjustment pattern displayed.

#### ● To adjust flickering, screen position, and screen size automatically

##### Procedure

1. Display Pattern 1 in full screen on the monitor using the “Screen adjustment pattern files”.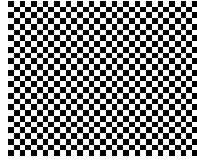

2. Choose “Screen” from the Adjustment menu, and press ENTER.
3. Choose “Analog Adjustment” from “Screen”, and press ENTER.
4. Choose “Auto Adjustment” from “Analog Adjustment”, and press ENTER.

Flickering, screen position, and screen size are corrected by the auto-adjustment function.

If the screen is not displayed correctly even after performing Auto Adjustment, perform the adjustments according to the procedures on the following pages. If the screen is displayed correctly, go to [“5. Adjusting color gradation.”](#) (page 15).

### 4. Choose “Screen” from the Adjustment menu to perform advanced adjustments.

Adjust the clock, phase and position, in this order.

#### ● To eliminate vertical bars


##### Procedure

1. Choose “Analog Adjustment” from “Screen”, and press ENTER.
2. Choose “Clock” from “Analog Adjustment”, and press ENTER.

The “Clock” menu appears.

3. Adjust the clock with ▲ or ▼ to eliminate vertical bars.  
Press ▲ or ▼ slowly so as not to miss the adjustment point.
4. Press ENTER to exit.

When blurring, flickering or bars appear on the screen after adjustment, proceed to “To remove flickering or blurring”.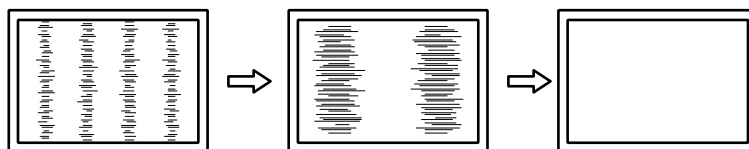

## ● To remove flickering or blurring

### **Procedure**


1. Choose “Analog Adjustment” from “Screen”, and press ENTER.
2. Choose “Phase” from “Analog Adjustment”, and press ENTER.

The “Phase” menu appears.

3. Adjust the phase with ▲ or ▼ to minimize flickering or blurring.
4. Press ENTER to exit.

When vertical bars appear on the screen after adjustment, go back to “To eliminate vertical bars”.

(Clock → Phase → Position)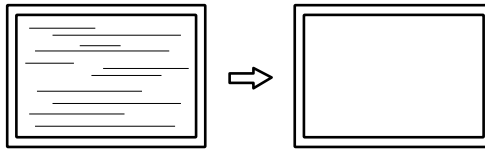

---

### **Attention**

- Flickering or blurring may not be eliminated depending on your PC or graphics board.
- 

## ● To correct the screen position

---

### **Note**


- Since the number of pixels and the pixel positions are fixed on the LCD monitor, only one position is provided to display images correctly. The position adjustment is made to shift an image to the correct position.
- 

### **Procedure**

1. Choose “Analog Adjustment” from “Screen”, and press ENTER.
2. Choose “Hor. Position” or “Ver. Position” from “Analog Adjustment”, and press ENTER.

The “Hor.Position” or “Ver.Position” menu appears.

3. Adjust the position with ▲ or ▼.
4. Press ENTER to exit.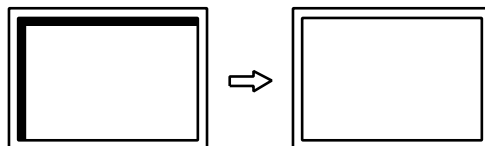

5. Close Pattern 1.


## 5. Adjusting color gradation.

### ● To adjust color gradation automatically

Every color gradation (0 to 255) can be displayed by adjusting the signal output level.

#### **Procedure**

1. Display Pattern 2 in full screen on the monitor using the “Screen adjustment pattern files”.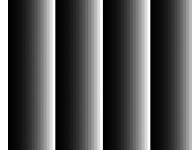


2. Choose “Screen” from the Adjustment menu, and press ENTER.
3. Choose “Analog Adjustment” from “Screen”, and press ENTER.
4. Choose “Range Adjustment” from “Analog Adjustment”, and press ENTER.  
The output range is set automatically.
5. Close Pattern 2.

## 2-5. Selecting Screen Size

The image with the resolution other than the recommended resolution is displayed in full screen automatically. You can change the screen size by using the “Screen Size” function from “Screen”.

Example: Image size 1280 × 1024

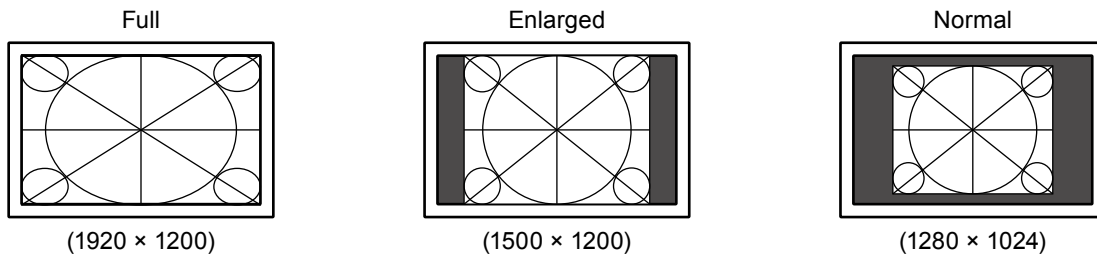#### **Procedure**

1. Select “Screen” from the Adjustment menu, and press ENTER.
2. Select “Screen Size” from “Screen”, and press ENTER.
3. Select “Full”, “Enlarged”, or “Normal” with ▲ or ▼.

Setting	Function
Full	Displays an image in full screen. Images are distorted in some cases because the vertical rate is not equal to the horizontal rate.
Enlarged	Displays an image in full screen. In some cases, a blank horizontal or vertical border appears to equalize the vertical rate and the horizontal rate.
Normal	Displays images with the specified resolution.

4. Press ENTER to exit.

# Chapter 3 Setting Monitor

## 3-1. Skipping Unused Display Modes

This function selects / deselects modes that should be skipped.

This setting is suitable when you are using limited display modes and you do not want to change the customized display state.

### **Procedure**

1. Select "Tools" from the Adjustment menu, and press ENTER.
2. Select "Mode Skip" from "Tools", and press ENTER.
3. Use ▲ or ▼ to select the mode to change, and press ENTER.
4. Select "Skip" or "-" with ▲ or ▼.
5. Press ENTER to exit.

---

### **Attention**

- You cannot set all the modes disable. Set at least one mode to "-".
- 

## 3-2. Setting Language

The language for the Setting menu and messages can be selected.

### **Selectable languages**

English, German, French, Spanish, Italian, Swedish, Japanese, Simplified Chinese, Traditional Chinese

### **Procedure**

1. Select "Menu Settings" from the Adjustment menu, and press ENTER.
2. Select "Language" from "Menu Settings", and press ENTER.
3. Select a language with ▲ or ▼.
4. Press ENTER to exit.

## 3-3. Setting Adjustment Menu Display

You can change the display position of the Adjustment menu.

### **Procedure**

1. Select "Menu Settings" from the Adjustment menu, and press ENTER.
2. Select "Menu Position" from "Menu Settings", and press ENTER.
3. Select the position with ▲ or ▼.
4. Press ENTER to exit.

## 3-4. Showing and hiding the EIZO logo

You can choose to show or hide the display of the EIZO logo.

### Procedure

1. Press  $\text{\textcircled{P}}$  to turn off the monitor.
2. Hold down MODE while pressing  $\text{\textcircled{P}}$  for at least two seconds to turn on the monitor.  
The “Optional Settings” menu appears.
3. Select “Logo” from “Optional Settings”, and press ENTER.
4. Select “On” or “Off” with  $\blacktriangle$  or  $\blacktriangledown$ , and press ENTER.
5. Select “Finish” with  $\blacktriangle$  or  $\blacktriangledown$ .
6. Press ENTER.  
The “Optional Settings” menu is closed.

## 3-5. Locking Operation Buttons

The settings can be locked to prevent them being changed.

### Procedure

1. Press  $\text{\textcircled{P}}$  to turn off the monitor.
2. Hold down MODE while pressing  $\text{\textcircled{P}}$  for at least two seconds to turn on the monitor.  
The “Optional Settings” menu appears.
3. Select “Key Lock” from “Optional Settings”, and press ENTER.
4. Select “Off”, “Menu”, or “All” with  $\blacktriangle$  or  $\blacktriangledown$ , and press ENTER.

Item	Buttons that can be locked
Off (Default setting)	None (All buttons are enabled)
Menu	ENTER button
All	All buttons excluding $\text{\textcircled{P}}$

5. Select “Finish” with  $\blacktriangle$  or  $\blacktriangledown$ .
6. Press ENTER.  
The “Optional Settings” menu is closed.

## 3-6. Enabling/Disabling DDC Communication

This function allows you to enable/disable the DDC communication. Normally, turn it on during use. (This is different from the DDC/CI communication.)

### Procedure

1. Press  $\text{\textcircled{P}}$  to turn off the monitor.
2. Hold down MODE while pressing  $\text{\textcircled{P}}$  for at least two seconds to turn on the monitor.  
The “Optional Settings” menu appears.
3. Select “DDC” from “Optional Settings”, and press ENTER.
4. Select “On” or “Off” with  $\blacktriangle$  or  $\blacktriangledown$ , and press ENTER.
5. Select “Finish” with  $\blacktriangle$  or  $\blacktriangledown$ .
6. Press ENTER.  
The “Optional Settings” menu is closed.

## 3-7. Resetting to Default

---

There are two different Resets: Color Reset for only resetting the color adjustments to the default settings and All Reset for resetting all settings to the defaults.

---

**Attention**

- After resetting, you cannot undo the operation.

---

**Note**

- Regarding the default setting, see [“6-7. Main Default Settings” \(page 34\)](#).
- 

### ● Resetting the color adjustments

Only the color adjustment value for the currently selected mode can be reset to the default settings.

**Procedure**

1. Select “Color” from the Adjustment menu, and press ENTER.
2. Select “Color Reset” from “Color”, and press ENTER.
3. Select “Execute” with ▲ or ▼.
4. Press ENTER.

The color adjustments revert to the default settings.

### ● Resetting all adjustments to the default settings

All settings are reset to the default settings (except the “Optional Settings” menus).

**Procedure**

1. Select “Tools” from the Adjustment menu, and press ENTER.
2. Select “All Reset” from “Tools”, and press ENTER.
3. Select “Execute” with ▲ or ▼.
4. Press ENTER.

All settings excluding the “Optional Settings” menus are reset to the default settings.

# Chapter 4 Power Saving Functions

## 4-1. Setting Power Save

The monitor can be set to the power saving mode according to the PC status. When the monitor has shifted to the power saving mode, the images on the screen are not displayed.

### Attention

- Turning off the main power switch or unplugging the power cord completely shuts off power supply to the monitor.
- Devices connected to the USB downstream port work even when the monitor is in power saving mode or when the power button of the monitor is Off. Therefore, power consumption of the monitor varies with connected devices even in the power saving mode.
- Five seconds before the monitor enters the power saving mode, it displays a preliminary message.

### Procedure

1. Select "PowerManager" from the Adjustment menu, and press ENTER.
2. Select "Power Save" from "PowerManager", and press ENTER.
3. Select "Off" and either "High" or "Low" with ▲ or ▼.

Setting	Function
High	Turns off the backlight. The power saving effect is the greatest.
Low	Turns down the backlight to the lowest setting. The time until the operation stabilizes after exiting the Power Save mode next time can be reduced.

4. Press ENTER to exit.

### Power Saving System

#### At DVI signal input

This monitor complies with the "DVI DMPM" standard.

#### At DisplayPort signal input

This monitor complies with the "DisplayPort Standard V1.1a".

The monitor enters the power saving mode after five seconds in connection with the PC setting.

PC	Monitor	Power indicator
On	Operation mode	Green
Power saving mode	Power saving mode	Orange

## 4-2. Adjusting the Brightness of Indicator

The brightness of the power indicator (green) when the screen is displayed can be set. (The power indicator is set to light up when the power is switched on, and set to 4 by default.)

### Procedure

1. Select "PowerManager" from the Adjustment menu, and press ENTER.
2. Select "Indicator" from "PowerManager", and press ENTER.
3. Select the indicator brightness "Off" or in the range from 1 to 7 with ▲ or ▼ as desired.
4. Press ENTER to exit.

## 4-3. Setting DisplayPort Power Save

---

The power setting of the DisplayPort of the monitor can be switched in the power saving mode.

### **Attention**

- When “On” is set, the display position on the window may not be positioned correctly when the monitor power is turned on or off. In such a case, turn the setting to “Off”.
- 

### **Procedure**

1. Press  $\odot$  to turn off the monitor.
2. Hold down MODE while pressing  $\odot$  for at least two seconds to turn on the monitor.  
The “Optional Settings” menu appears.
3. Select “DP Power Save” from “Optional Settings”, and press ENTER.
4. Select “On” or “Off” with  $\blacktriangle$  or  $\blacktriangledown$ , and press ENTER.
5. Select “Finish” with  $\blacktriangle$  or  $\blacktriangledown$ .
6. Press ENTER.  
The “Optional Settings” menu is closed.

# Chapter 5 Troubleshooting

If a problem still remains after applying the suggested remedies, contact your local EIZO representative.

## 5-1. No Picture

Problem	Possible cause and remedy
<p><b>1. No picture</b></p> <ul style="list-style-type: none"> <li>Power indicator does not light.</li> </ul> <hr/> <ul style="list-style-type: none"> <li>Power indicator is lighting green.</li> </ul> <hr/> <ul style="list-style-type: none"> <li>Power indicator is lighting orange.</li> </ul> <hr/> <ul style="list-style-type: none"> <li>Power indicator is flashing orange and green.</li> </ul>	<ul style="list-style-type: none"> <li>Check whether the power cord is connected properly.</li> <li>Turn the main power switch on.</li> <li>Press .</li> <li>Turn off the main power, and then turn it on again a few minutes later.</li> </ul> <hr/> <ul style="list-style-type: none"> <li>Increase “Brightness” and/or “Gain” in the Adjustment menu (see <a href="#">“Performing advanced adjustments” (page 8)</a>).</li> </ul> <hr/> <ul style="list-style-type: none"> <li>Switch the input signal with SIGNAL.</li> <li>Move the mouse or press any key on the keyboard.</li> <li>Check whether the PC is turned on.</li> </ul> <hr/> <ul style="list-style-type: none"> <li>The device that is connected using DisplayPort have a problem. Solve the problem, turn off the monitor, and then turn it on again. Refer to the User’s Manual of the output device for further details.</li> </ul>
<p><b>2. The message below appears.</b></p> <hr/> <ul style="list-style-type: none"> <li>This message appears when no signal is input. Example:</li> </ul> <div data-bbox="260 1093 662 1355" data-label="Image"> </div> <hr/> <ul style="list-style-type: none"> <li>The message shows that the input signal is out of the specified frequency range. (Such signal frequency is displayed in magenta.) Example:</li> </ul> <div data-bbox="260 1496 662 1848" data-label="Image"> </div> <p>fD: Dot clock (Displayed only during digital signal input) fH: Horizontal scan frequency fV: Vertical scan frequency</p>	<p>This message appears when the signal is not input correctly even when the monitor functions properly.</p> <hr/> <ul style="list-style-type: none"> <li>The message shown left may appear, because some PCs do not output the signal soon after power-on.</li> <li>Check whether the PC is turned on.</li> <li>Check whether the signal cable is connected properly.</li> <li>Switch the input signal with SIGNAL.</li> </ul> <hr/> <ul style="list-style-type: none"> <li>Check whether the PC is configured to meet the resolution and vertical scan frequency requirements of the monitor (see <a href="#">“Compatible Resolutions”</a> in the <a href="#">“Instructions for Use”</a>).</li> <li>Reboot the PC.</li> <li>Select an appropriate setting using the graphics board’s utility. Refer to the manual of the graphics board for details.</li> </ul>

## 5-2. Imaging Problems (for both digital and analog)

Problem	Possible cause and remedy
1. The screen is too bright or too dark.	<ul style="list-style-type: none"> <li>Use "Brightness" in the Adjustment menu to adjust it. (The LCD monitor backlight has a limited life span. When the screen becomes dark or begins to flicker, contact your local EIZO representative.)</li> </ul>
2. Characters are blurred.	<ul style="list-style-type: none"> <li>Check whether the PC is configured to meet the resolution and vertical scan frequency requirements of the monitor (see "Compatible Resolutions" in the Setup Manual.).</li> </ul>
3. Afterimages appear.	<ul style="list-style-type: none"> <li>Afterimages are particular to LCD monitors. Avoid displaying the same image for a long time.</li> <li>Use the screen saver or power saving function to avoid displaying the same image for extended periods of time.</li> </ul>
4. Green/red/blue/white dots or defective dots remain on the screen.	<ul style="list-style-type: none"> <li>This is due to LCD panel characteristics and is not a failure.</li> </ul>
5. Interference patterns or pressure marks remain on the screen.	<ul style="list-style-type: none"> <li>Leave the monitor with a white or black screen. The symptom may disappear.</li> </ul>

## 5-3. Imaging Problems (for analog only)

Problem	Possible cause and remedy
1. Display position is incorrect. 	<ul style="list-style-type: none"> <li>Use "Position" in the Adjustment menu to correct the image position (see <a href="#">"To correct the screen position"</a> (page 14)).</li> <li>If the problem persists, use the graphics board's utility if available to change the display position.</li> </ul>
2. Vertical bars appear on the screen or a part of the image is flickering. 	<ul style="list-style-type: none"> <li>Use "Clock" in the Adjustment menu to adjust it (see <a href="#">"To eliminate vertical bars"</a> (page 13)).</li> </ul>
3. Whole screen is flickering or blurring. 	<ul style="list-style-type: none"> <li>Use "Phase" in the Adjustment menu to adjust it (see <a href="#">"To remove flickering or blurring"</a> (page 14)).</li> </ul>

## 5-4. Other Problems

Problem	Possible cause and remedy
<b>1. The adjustment menu does not appear.</b>	<ul style="list-style-type: none"><li>• Check whether the operation lock function is active (see <a href="#">“3-5. Locking Operation Buttons” (page 17)</a>).</li></ul>
<b>2. No mode menu is displayed.</b>	<ul style="list-style-type: none"><li>• Check whether the operation lock function is active (see <a href="#">“3-5. Locking Operation Buttons” (page 17)</a>).</li></ul>
<b>3. The monitor connected with the USB cable is not detected. / USB devices connected to the monitor does not work.</b>	<ul style="list-style-type: none"><li>• Check whether the USB cable is connected correctly (see <a href="#">“6-5. Making Use of USB (Universal Serial Bus)” (page 32)</a>).</li><li>• Change the USB port to another one. If the PC or peripheral devices works correctly by changing the USB port, contact your local EIZO representative. (Refer to the manual of the PC for details.)</li><li>• Reboot the PC.</li><li>• If the peripheral devices work correctly when the PC and peripheral devices are connected directly, please contact your local EIZO representative.</li><li>• Check whether the PC and its OS are USB compliant. (For USB compliance of the respective devices, consult their manufacturers.)</li><li>• For Windows, check the PC’s BIOS setting for USB. (Refer to the manual of the PC for details.)</li></ul>

# Chapter 6 Reference

## 6-1. Attaching Arm

An optional arm (or an optional stand) can be attached by removing the stand section. Please refer to our web site for the corresponding optional arm (or optional stand). <http://www.eizo.com>

### Attention

- When attaching an arm or stand, follow the instructions of their User's Manual.
- When using another manufacturer's arm or stand, confirm the following in advance and select one conforming to the VESA standard.
  - Clearance between the screw holes: 100 mm × 100 mm
  - Strong enough to support weight of the monitor unit (except the stand) and attachments such as cables.
- When using another manufacturer's arm or stand, use the following screws to secure it.
  - Screws fixing the stand to the monitor
- When using an arm or stand, attach it to meet the following tilt angles of the monitor.
  - Up 45°, down 45°
- Connect the cables after attaching an arm or stand.
- Do not move the removed stand up and down. Doing so may result in injury or device damage.
- The monitor and arm or stand are heavy. Dropping them may result in injury or equipment damage.
- When setting the monitor to the portrait style, turn the monitor screen 90° in clockwise direction.

### Procedure


**1. Lay the LCD monitor on a soft cloth spread over on a stable surface with the panel surface facing down.**

**2. Remove the stand.**

Prepare a screwdriver. Loosen the screws (four) securing the unit and the stand with the screwdriver.

**3. Attach the arm or stand to the monitor.**

Secure the monitor to the arm or stand using the screws fixing the stand to the monitor.

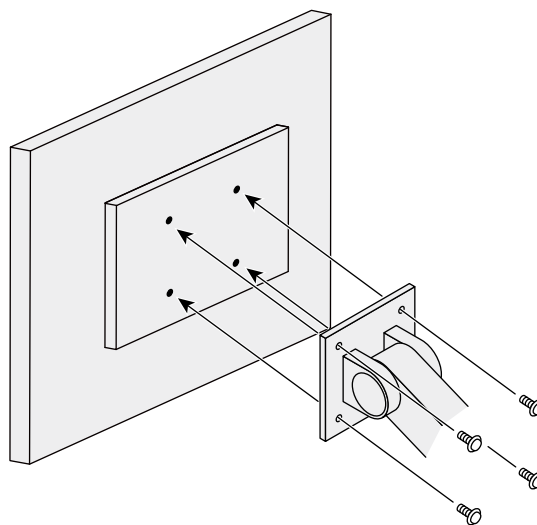
## 6-2. Detaching/Attaching Stand Base


### Detaching procedure

The stand base that was attached during setup can be detached using the following procedures.

- 1. Lay the LCD monitor on a soft cloth spread over on a stable surface with the panel surface facing down.**
- 2. Raise the lever on the bottom of the stand base.**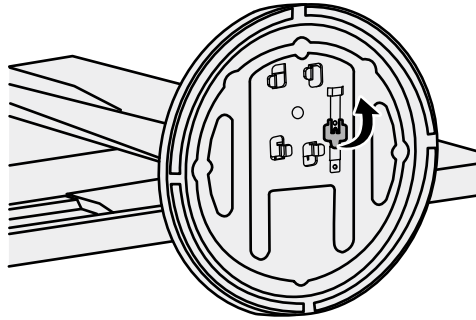


- 3. Turn the stand base in the counterclockwise direction to detach it.**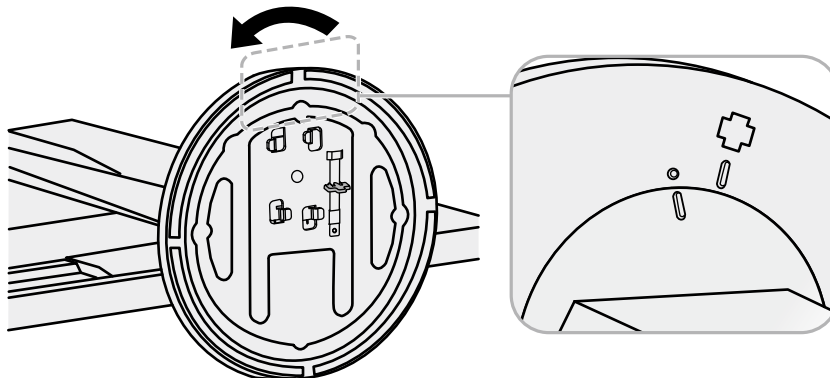


## Attaching procedure

The once removed the stand base can be attached to the monitor again with the following procedures.


- 1. Lay the LCD monitor on a soft cloth spread over on a stable surface with the panel surface facing down.**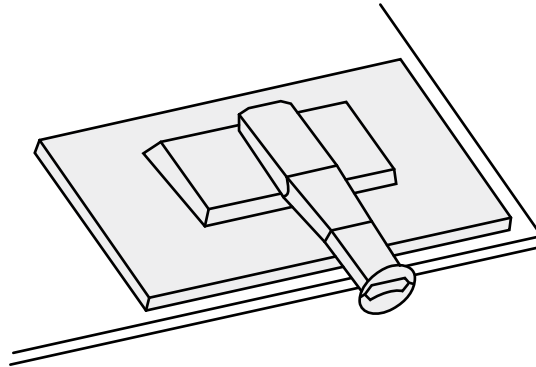

- 2. Attach the stand base onto the stand brace.**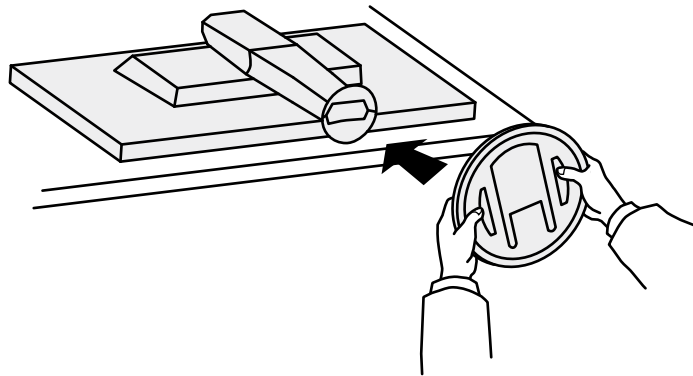

**3. Turn the stand base in the clockwise direction until it clicks.**

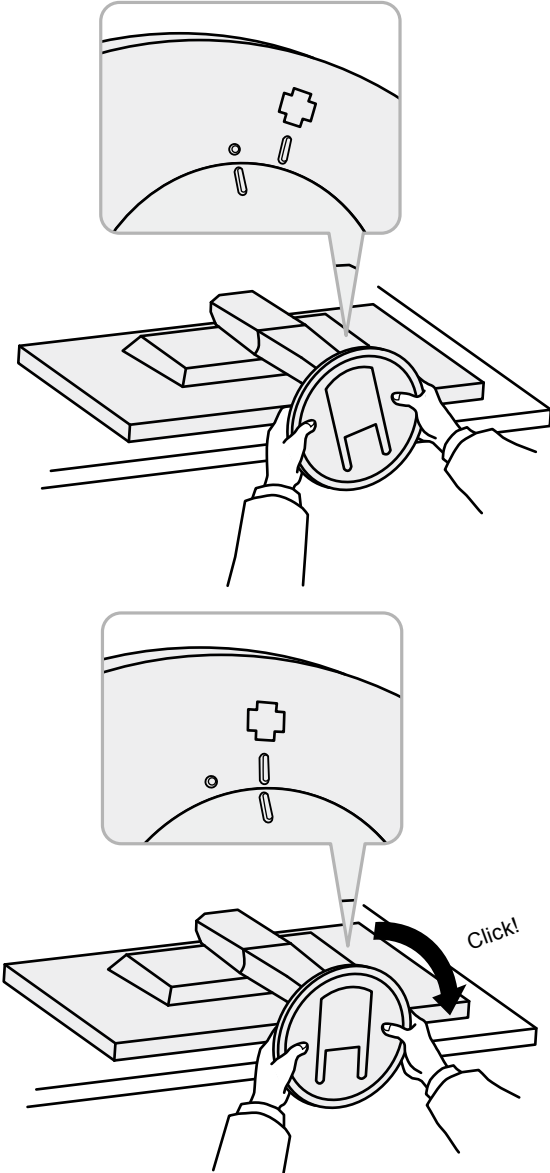
## 6-3. Attaching/Detaching Cable Holder


A cable holder is supplied with this product. Use the holder to organize the cables connected to the monitor.

### Attaching procedure

1. Pass the cables through the cable holder.
2. Close the opening in the cable holder.

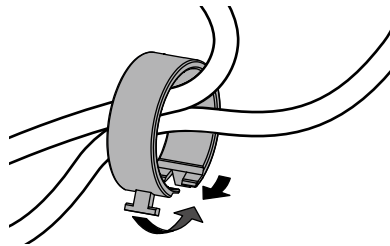
3. While holding the cable holder closed, attach it onto the stand.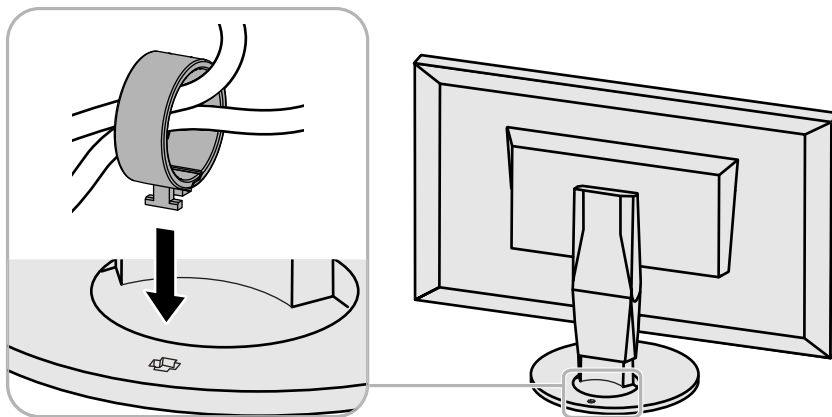


#### Note

- The cable holder can be inserted either perpendicular or parallel to the stand. Change the direction of the cable holder according to the conditions of the cables.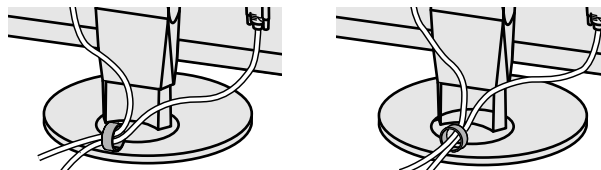

**Detaching procedure**


- 1.** Close the opening in the cable holder.
- 2.** While holding the cable holder closed, remove it from the stand.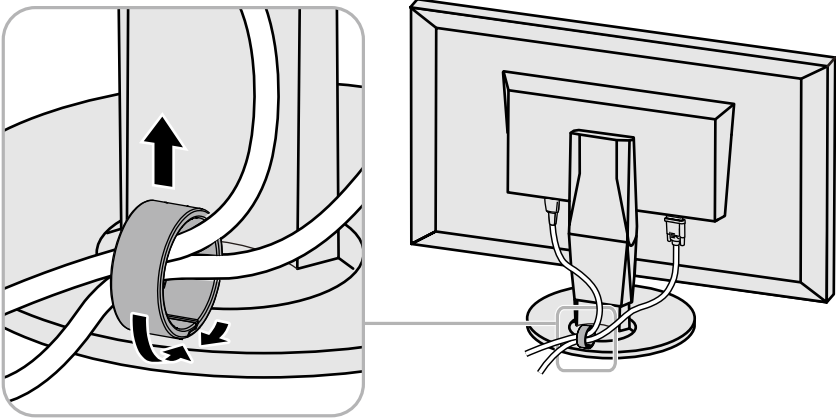

## 6-4. Connecting Multiple PCs

The product has multiple connections to PCs and allows you to switch to one of the connections for display.

### Connection examples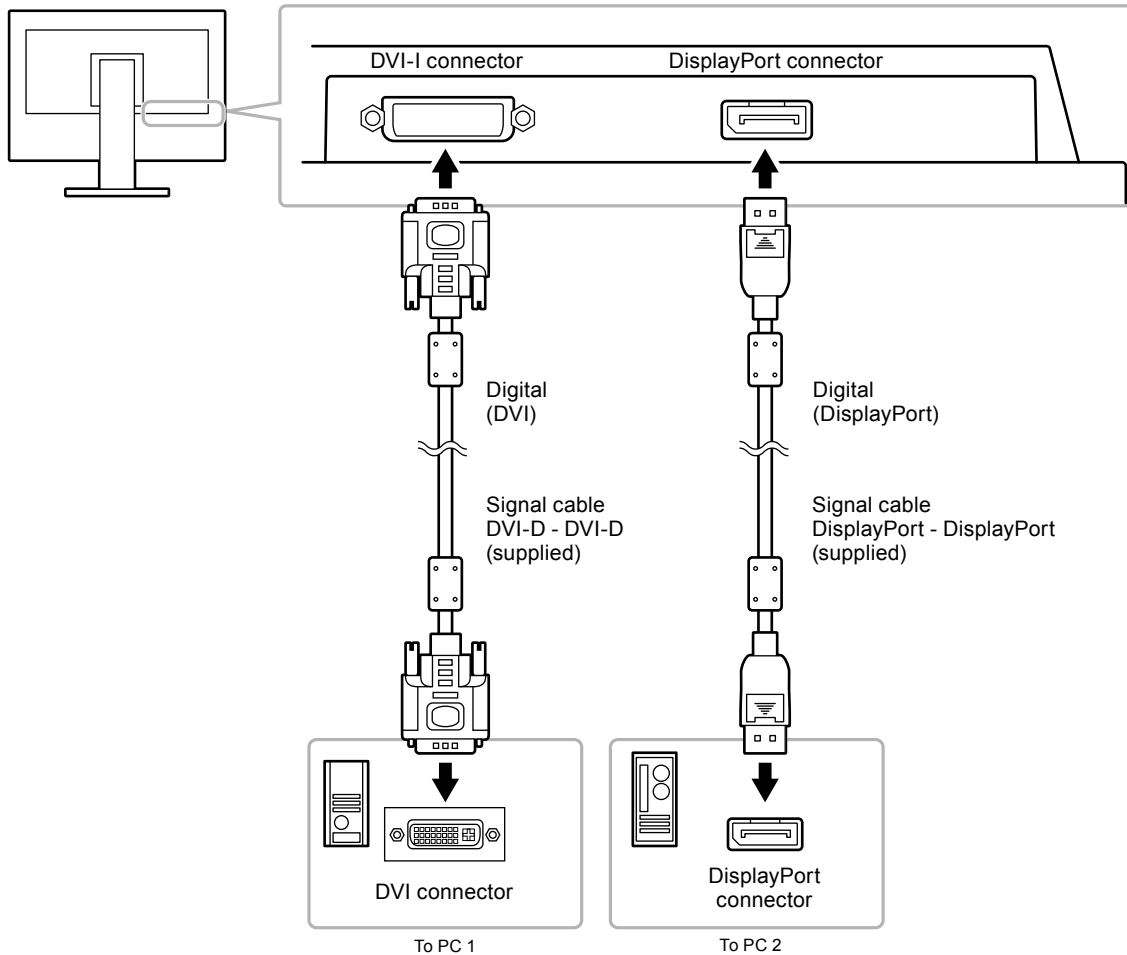


### ● Switching among input signals

#### Procedure

The input signal switches each time SIGNAL is pressed.

When the signal is switched, the active input port name appears at the top right corner of the screen.

(Example)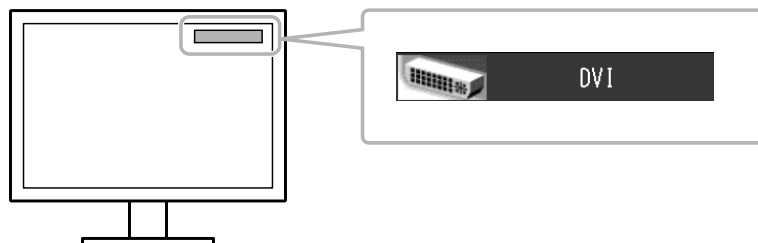

## ● Setting input signal switching procedure

Setting	Function
Auto	The monitor recognizes the connector through which PC signals are input. When a PC is turned off or enters the power saving mode, the monitor automatically displays another signal.
Manual	The monitor displays the signal from the selected connector despite the signal input. Select an active input signal with SIGNAL.

### **Procedure**

1. Select "Tools" from the Adjustment menu, and press ENTER.
2. Select "Input Selection" from "Tools", and press ENTER.
3. Select "Auto" or "Manual" with ▲ or ▼.
4. Press ENTER to exit.

---

### **Note**

- If you select "Auto", the monitor enters the power saving mode after all the PCs have entered the power saving mode or have been switched off.
-

## 6-5. Making Use of USB (Universal Serial Bus)

This monitor has a USB hub. It works as a USB hub when connected to a USB-compatible PC, allowing the connection of peripheral USB devices.

### ● Required system environment


- A PC equipped with a USB port
- Windows 8/7/Vista/XP/2000 or Mac OS 9.2.2 and Mac OS X 10.2 or later
- USB cable

#### **Attention**

- This monitor may not work depending on PC, OS or peripheral devices to be used. For USB compatibility of peripheral devices, contact their manufacturers.
- Devices connected to the USB downstream port work when the monitor is in power saving mode or when the power button of the monitor is Off.
- When the main power of the monitor is Off, a device connected to the USB downstream port will not operate.

### ● Connection procedure (Setup of USB function)

1. Connect the monitor first to a PC using the signal cable, and run the PC.
2. Connect the USB cable between the USB downstream port of the PC and the USB upstream port of the monitor.  
The USB hub function is set up automatically upon connection of the USB cable.
3. Connect the peripheral USB devices to the USB downstream port of the monitor.

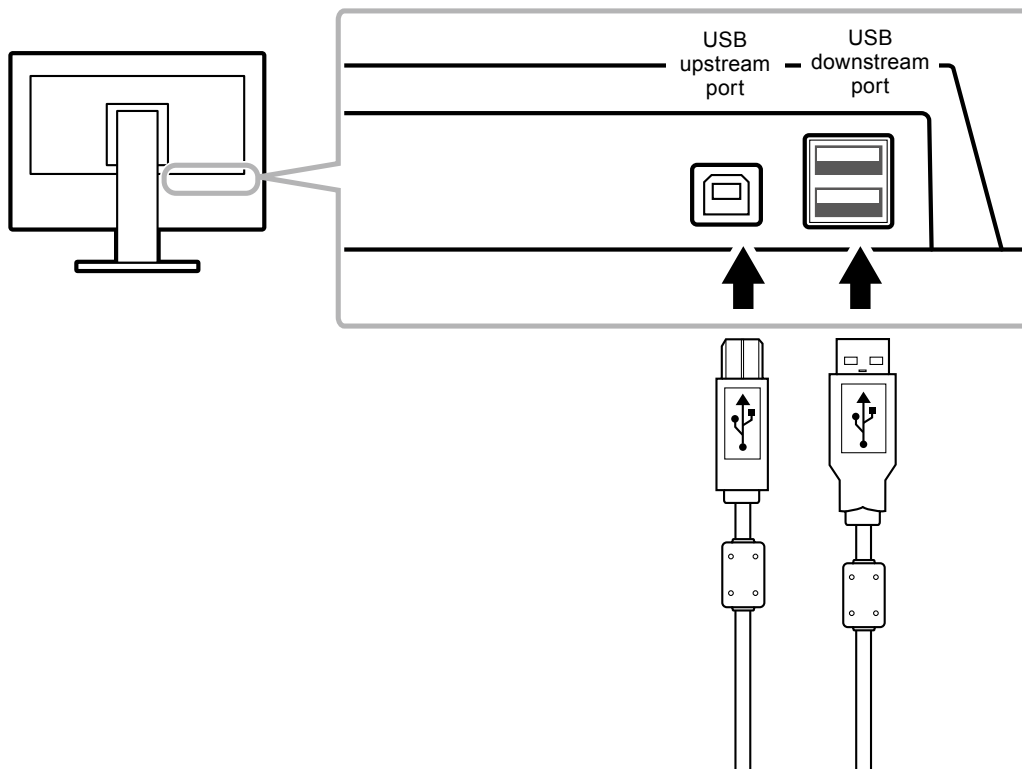## 6-6. Showing Monitor Information

### ● Displaying signal information


The information about the current input signals can be displayed.

#### **Procedure**

1. Select “Tools” from the Adjustment menu, and press ENTER.
2. Select “Signal Info” from “Tools”, and press ENTER.

The “Signal Info” appears.

(Example)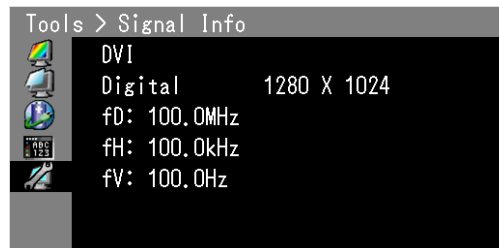

### ● Showing monitor information


The information about the monitor can be displayed.

#### **Procedure**

1. Select “Tools” from the Adjustment menu, and press ENTER.
2. Select “Monitor Info” from “Tools”, and press ENTER.

The “Monitor Info” appears.

The model name, serial number, firmware version, and usage time can be displayed.

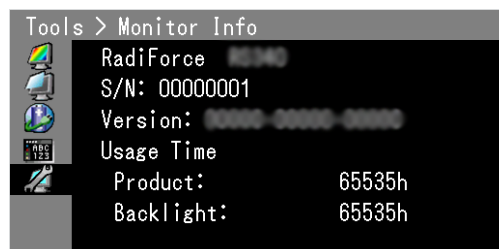---

#### **Attention**

- When you turn on the monitor for the first time, the usage time may be more than 0 due to a factory inspection.
-

## 6-7. Main Default Settings

**CAL Switch Mode:** The default display setting is 1-DICOM.

Mode	Brightness	Temperature	Gamma value
1-DICOM	180 cd/m <sup>2</sup>	7,500 K	DICOM
2-Custom	Approx. 300 cd/m <sup>2</sup>	7,500 K	2.2
3-CAL	250 cd/m <sup>2</sup>	6,500 K	DICOM
4-Text	Approx. 100 cd/m <sup>2</sup>	7,500 K	2.2

### Others

Power Save	High
DP Power Save <sup>*1</sup>	Off
Input Selection	Auto
Logo <sup>*1</sup>	Off
Key Lock <sup>*1</sup>	Off
Screen Size	Normal
Menu Position	Center
Language	English

<sup>\*1</sup> These menus cannot be initialized with "All Reset" (see [page 18](#)).

## 6-8. Preset Timing

The following table shows factory preset analog signal timing.

### Attention

- Display position may be deviated depending on the PC connected, which may require screen adjustment using Adjustment menu.
- If a signal other than those listed in the table is input, adjust the screen using the Adjustment menu. However, screen display may still be incorrect even after the adjustment.
- When interlace signals are used, the screen cannot be displayed correctly even after screen adjustment using the Adjustment menu.

Resolution	Frequency	
	Horizontal: kHz	Vertical: Hz
640 × 480	31.47	59.94
720 × 400	31.47	70.09
800 × 600	35.16	56.25
800 × 600	37.88	60.32
1024 × 768	48.36	60.00
1024 × 768	56.48	70.07
1280 × 960	60.00	60.00
1280 × 1024	63.98	60.02
1600 × 1200	75.00	60.00
1680 × 1050	65.29	59.95
1920 × 1200	74.04	59.95

# LIMITED WARRANTY

## LIMITED WARRANTY

EIZO Corporation (hereinafter referred to as “**EIZO**”) and distributors authorized by EIZO (hereinafter referred to as the “**Distributors**”) warrant, subject to and in accordance with the terms of this limited warranty (hereinafter referred to as the “**Warranty**”), to the original purchaser (hereinafter referred to as the “**Original Purchaser**”) who purchased the product specified in this document (hereinafter referred to as the “**Product**”) from EIZO or Distributors, that EIZO and Distributors shall, at their sole discretion, either repair or replace the Product at no charge if the Original Purchaser becomes aware within the Warranty Period (defined below) that (i) the Product malfunctions or is damaged in the course of normal use of the Product in accordance with the description in the instruction manual attached to the Product (hereinafter referred to as the “**User’s Manual**”), or (ii) the LCD panel and brightness of the Product cannot maintain the recommended brightness specified in the User’s Manual in the course of normal use of the Product in accordance with the description of the User’s Manual.

The period of this Warranty is five (5) years from the date of purchase of the Product (hereinafter referred to as the “**Warranty Period**”), provided that the brightness of the Product shall be warranted only if the Product has been used within the recommended brightness described in the User’s Manual and the Warranty Period for the brightness is limited to five (5) years from the date of purchase of the Product subject to the usage time being less than or equal to 15,000 hours (the brightness is 180 cd/m<sup>2</sup> and color temperature is 7500K). EIZO and Distributors shall bear no liability or obligation with regard to the Product in relation to the Original Purchaser or any third parties other than as provided under this Warranty.

EIZO and Distributors will cease to hold or store any parts of the Product upon expiration of seven (7) years after the production of such parts is discontinued. In repairing the monitor, EIZO and Distributors will use renewal parts which comply with our QC standards.

The Warranty is valid only in the countries or territories where the Distributors are located. The Warranty does not restrict any legal rights of the Original Purchaser.

Notwithstanding any other provision of this Warranty, EIZO and Distributors shall have no obligation under this Warranty whatsoever in any of the cases as set forth below:

- (a) Any defect of the Product caused by freight damage, modification, alteration, abuse, misuse, accident, incorrect installation, disaster, faulty maintenance and/or improper repair by third party other than EIZO and Distributors;
- (b) Any incompatibility of the Product due to possible technical innovations and/or regulations;
- (c) Any deterioration of the sensor;
- (d) Any deterioration of display performance caused by the deterioration of expendable parts such as the LCD panel and/or backlight, etc. (e.g. changes in color, changes in color uniformity, defects in pixels including burnt pixels, etc.);
- (e) Any deterioration of the Product caused by the use at higher brightness than the recommended brightness described in the User’s Manual;
- (f) Any defect of the Product caused by external equipment;
- (g) Any defect of the Product on which the original serial number has been altered or removed;
- (h) Any normal deterioration of the product, particularly that of consumables, accessories, and/or attachments (e.g. buttons, rotating parts, cables, User’s Manual, etc.); and
- (i) Any deformation, discoloration, and/or warp of the exterior of the product including that of the surface of the LCD panel.

To obtain service under the Warranty, the Original Purchaser must deliver the Product, freight prepaid, in its original package or other adequate package affording an equal degree of protection, assuming the risk of damage and/or loss in transit, to the local Distributor. The Original Purchaser must present proof of purchase of the Product and the date of such purchase when requesting services under the Warranty.

The Warranty Period for any replaced and/or repaired product under this Warranty shall expire at the end of the original Warranty Period.

EIZO OR DISTRIBUTORS ARE NOT RESPONSIBLE FOR ANY DAMAGE TO, OR LOSS OF, DATA OR OTHER INFORMATION STORED IN ANY MEDIA OR ANY PART OF ANY PRODUCT RETURNED TO EIZO OR DISTRIBUTORS FOR REPAIR.

EIZO AND DISTRIBUTORS MAKE NO FURTHER WARRANTIES, EXPRESSED OR IMPLIED, WITH RESPECT TO THE PRODUCT AND ITS QUALITY, PERFORMANCE, MERCHANTABILITY OR FITNESS FOR ANY PARTICULAR USE. IN NO EVENT SHALL EIZO OR DISTRIBUTORS BE LIABLE FOR ANY INCIDENTAL, INDIRECT, SPECIAL, CONSEQUENTIAL OR OTHER DAMAGE WHATSOEVER (INCLUDING, WITHOUT LIMITATION, DAMAGES FOR LOSS OF PROFIT, BUSINESS INTERRUPTION, LOSS OF BUSINESS INFORMATION, OR ANY OTHER PECUNIARY LOSS) ARISING OUT OF THE USE OR INABILITY TO USE THE PRODUCT OR IN ANY CONNECTION WITH THE PRODUCT, WHETHER BASED ON CONTRACT, TORT, NEGLIGENCE, STRICT LIABILITY OR OTHERWISE, EVEN IF EIZO OR DISTRIBUTORS HAVE BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. THIS EXCLUSION ALSO INCLUDES ANY LIABILITY WHICH MAY ARISE OUT OF THIRD PARTY CLAIMS AGAINST THE ORIGINAL PURCHASER. THE ESSENCE OF THIS PROVISION IS TO LIMIT THE POTENTIAL LIABILITY OF EIZO AND DISTRIBUTORS ARISING OUT OF THIS LIMITED WARRANTY AND/OR SALES.

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